

HIV SINGCLEAN[®] *(1/2)*

Antibody Urine Test Kit (Colloidal Gold Method)



SELF-TESTING



PRECISE



RAPID



SENSITIVE



HANGZHOU SINGCLEAN MEDICAL PRODUCTS CO.,LTD.



What is HIV?


HIV or Human Immunodeficiency Virus attacks and impairs immune cells, causing progressive weakness of the immune system and can further progress to a condition called AIDS.

The Importance of HIV Test

Many people don't know right away when they are infected with HIV because they look and feel healthy. You can have HIV for many years and not feel sick. The only way to know if you have HIV is to take a test.

Early detection of infection can effectively reduce the spread of HIV and early treatment can prevent progression to AIDS.

Window Period for HIV Test

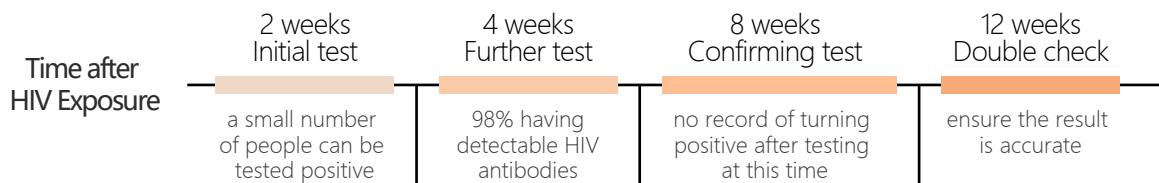

About Singclean HIV Test

Singclean® HIV-1/2 Antibody **Urine** Test Kit (Colloidal Gold Method) is used for the qualitative determination of human immunodeficiency virus type I (HIV-1) antibodies and type II (HIV-2) antibodies in human urine samples. It is suitable for auxiliary diagnosis of HIV infection and can be used for self-testing. An easy and convenient at-home HIV test, and can also be used for AIDS society, medical institution, medical clinics, CDC and so on.



AT HOME



MEDICAL INSTITUTION



CDC



AIDS SOCIETY

After infected with HIV, except in blood, HIV antibody will also appear in urine, so urine sample can be used for HIV test and it does not contain virus, which is a safer sample for testing.



SINGCLEAN®

■ People at Higher Risk of Getting HIV



Have anal, vaginal or oral sex without a condom



breastfeeding



using drugs



Share syringes, needles, cotton or other injection supplies

■ HIV IS NOT TRANSMITTED



CLOTHES TOWELS



TOILET SHOWER



INSECT BITES



FOOD DRINK UTENSILS

■ Singclean Advantages

Precise: high accuracy, the accuracy rate can reach 95%.

Rapid: just 15 mins to read the test result.

Painless: unlike collecting blood sample, detecting urine sample won't cause pain. No wound, no secondary infection.

Convenient: the test can be performed anytime, anywhere by yourself.

Privacy protection: at-home testing and can protect privacy of customer.






SINGCLEAN[®]

Specification

Format: Cassette 1 test/box 10tests/box 20 tests/box 50tests/box

Storage: Store at 4~30°C, avoid light and keep dry

shelf life: 24 months

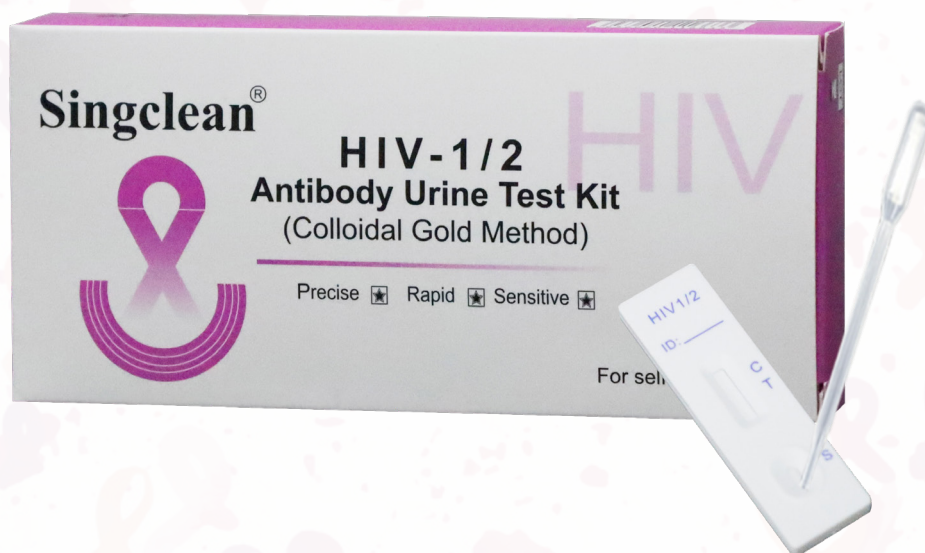

Test Procedure

1




Collect urine sample

2



Use the provided dropper to draw the urine sample

3

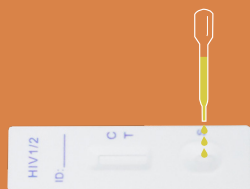

Add 3 drops of the urine sample vertically into the sample well (avoid air bubbles)

4



Waiting for results. Results should be read after 15 minutes.
The results after 20 minutes are not convincing



Interpretation of Results

Positive:


In addition to the presence of a C-line, if a T-line is present, the test indicates the presence of HIV-1/2 antibodies in the sample. The result is positive.

Negative:

If only the C line is present and there is no purple-red in the T line, HIV-1/2 antibodies are not detected in the specimen. The result is negative.

Invalid:

Control lines do not appear. Regardless of whether there is a T line, it is an invalid result.

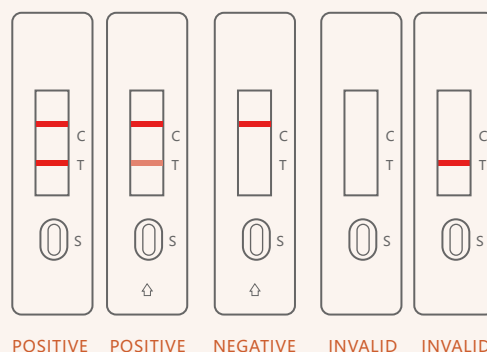

Limitations

1. This kit is a clinical auxiliary diagnostic product, which can rapidly detect human immunodeficiency virus type I (HIV-1) antibody and type II (HIV-2) antibody in urine. The test results are for clinical reference only and cannot be used alone as the basis for confirming or excluding cases. For the purpose of diagnosis, the test results should be used in conjunction with clinical examination, medical history and other examinations.
2. Due to the limitations of the method, a negative test result does not rule out the possibility of HIV infection in patients, especially those infected in the window period.
3. Samples undergoing antiviral infection treatment may have false negatives.
4. The test sample is urine, and other body fluids may be inaccurate.
5. The presence of interfering substances in the sample may lead to incorrect test results.



Singclean

Dedicated to Healthcare Improvement



You **cannot** get HIV from
hugging or touching

HANGZHOU SINGCLEAN MEDICAL PRODUCTS CO.,LTD.

ADDRESS: No.125(E),10th Street, Hangzhou Qiantang Area, Zhejiang,China

E-MAIL: Marketing@singclean.net Tel.: +86-571-63431868 Fax: 86-571-64341886

WEB ADDRESS: www.singclean.net